

HeelZup® Cloud

Single Patient Use Coverlet



The HeelZup® Cloud Coverlet helps to protect your Reusable HeelZup® Cloud Cushions by providing a moisture proof, friction and shear reducing barrier between the cushion and the user. The coverlet also raises infection control standards by providing each patient with a new, fresh out of the package surface. The HeelZup® Cloud Coverlet combines the cost effectiveness of reusable cushions with the infection control standards of a single patient use contact surface. The HeelZup® Cloud Coverlet is cost effective and easy to use!



Place the tapered end of the cushion in the deep pocketed end of the coverlet.



Wrap the coverlet across the top of the cushion and place the shallow pocketed end around the flat end of the cushion.



A snap on each strap holds the coverlet securely in place.



Join the snaps on the straps to the snaps on the bottom of the deep pocketed end of the coverlet.



The HeelZup® Cloud Cushion is ready for use!

Phone: 866.736.8484
Fax: 941.359.6939
www.heelzup.com

Intensive Therapeutics, Inc.

P.O. Box 488
Sarasota, FL
34230