

Bariatric No-Slip Wedge™

The Bariatric No-Slip Wedge enhances the ITI Patient Positioner Line by adding a highly reliable turning solution for Bariatric Patients. The Bariatric No-Slip Wedge™ is manufactured of a very firm, high-density foam core foam that will hold the torso in a turned position, while resisting the collapsing forces of heavier patient weights. We then add a layer of high-density visco-elastic foam to the patient contact surface of the Bariatric No-Slip Wedge™, which enhances comfort and pressure redistribution where it is needed most. The convex shape of the sides that face both the patient and caregiver create a more “Roll-Like” design, which improves shape retention while in use. The Bariatric No-Slip Wedge™ is an ideal way to enhance your pressure management and turning protocols for Bariatric Patients:

- Reliable way to maintain 30 degrees of lateral rotation for Bariatric Patients.
- Intended for Bariatric Patients weighing from 300 pounds to 650 pounds.
- Convex surfaces on the patient and caregiver sides make the wedge more substantial and less likely to collapse under heavier patients.
- Manufactured of high-density, anti-microbial, hypo-allergenic and latex-free foam.
- Covered with a skin friendly, moisture proof, friction and shear reducing cover on the patient contact side and a No-Slip fabric on the bottom.
- No-Slip fabric on the bottom prevents the wedge from migrating away from the patient. Helps to keep patients elevated for the full duration of the turning schedule.



U.S. Patent # 7,240,384

Phone: 866.736.8484
Fax: 941.359.6939
www.heelzup.com

Intensive Therapeutics, Inc.

P.O. Box 488
Sarasota, FL
34230

Specifications:



Bariatric No-Slip Wedge Reusable:

Manufactured of high-density, anti-microbial, hypoallergenic foam. The dimensions are 26" W x 12" Deep x 8" Tall. Upholstered with a moisture-proof, reusable, coated nylon fabric cover, which is friction and shear reducing. No-Slip fabric is used on the bottom to prevent migration away from the patient. The cover can be cleaned with standard cleaning procedures, using properly diluted solutions. The No-Slip Wedge meets or exceeds the specifications in California Upholstery Provisions as stated in Technical Bulletin #117.



This profile photo shows the firm inner core of the Bariatric No-Slip Wedge™, covered by the soft, pressure reducing Visco-Elastic foam layer. The Disposable Bare Foam Bariatric No-Slip Wedge comes without the fabric cover.

Infection Control Cover Features:



Waterfall Flap covers the top half of the zipper enclosure.



Raised border behind the bottom zipper prevents fluid infiltration.



Heat sealed seams available on optional Dartex® cover.

Ordering Information:

NSW-RB Bariatric No-Slip Wedge, Reusable, Nylon Cover, No-Slip Bottom
NSW-DB Bariatric No-Slip Wedge, Disposable Bare Foam



Warranty:

This warranty applies only to defects in structure, workmanship and materials and only to damages arising from normal hospital usage on a non-prorated basis for a period of Six (6) months from the date of purchase. Damages arising from abnormal use such as those caused by needle punctures, burns, chemicals, negligent use or improper care or cleaning are excluded from coverage. Manufacturer will pay shipping charges incurred in connection with this warranty.