


Intensive Therapeutics, Inc.

866.736.8484
www.heelzup.com
info@heelzup.com

P.O. Box 488
Sarasota, FL 34230

Time2Turn™ Visual Cue Patient Turning Reminder System

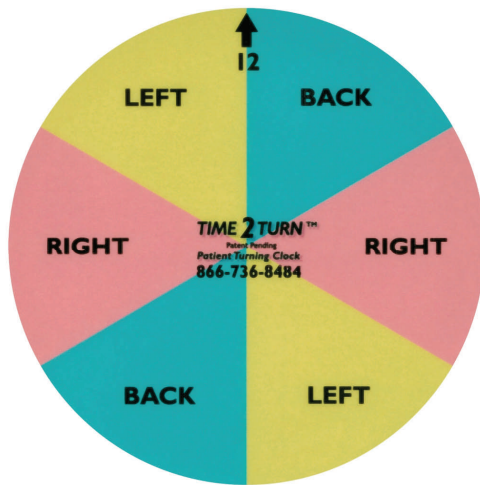

Patent Pending

Time2Turn™ is a translucent clock face overlay that turns your existing clock into a visual cue reminder system for patient turning. Each color coded two hour section provides a clear indication of when the next turn should occur and whether the turn should be to the “BACK”, “RIGHT” or “LEFT” position. Time2Turn™ is easily implemented, cost effective and always visible so every care provider on every shift is perpetually reminded of when and how to turn.

The Time2Turn™ clock overlay is made of transparent static cling film. The film applies and sticks to the clock face with no adhesive, so removal is as easy as application!

The Time2Turn™ clock overlay stays in place until you decide to remove it. The see-through film perpetually reminds the staff of when and how to turn their patients.

Ordering Information

T2T-CLK Translucent Static Cling Clock Face Overlay, 6.25” Diameter, one size fits all.

Instructions for Use:

1. Peel the Time2Turn™ Clock Overlay away from it's paper backing.
2. Align the arrow at the top of the clock overlay with the 12 o'clock position on the clock. Make sure that the overlay is centered properly over the clock face.
3. Hang the clock on the wall and begin use. Observe the color and cue where you find the hour hand. For example at 3:00 AM the patient should be turned to the right.

