
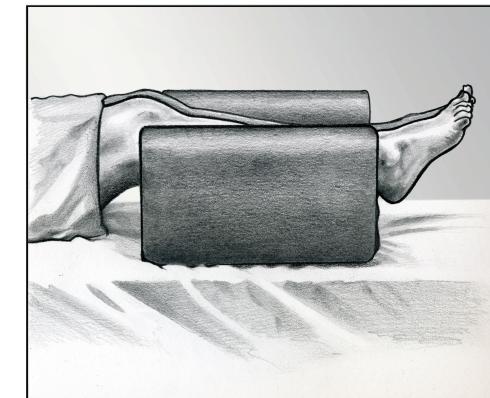


Intensive Therapeutics, Inc.
HeelZup® Heel Elevating Cushion - Covered Reusable Version Instruction Sheet

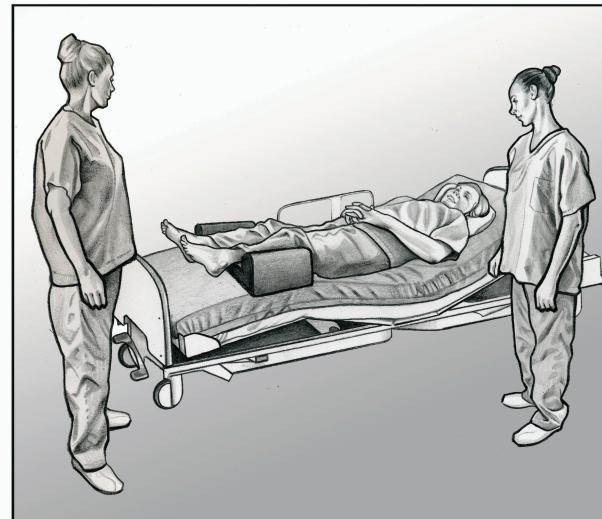

U.S. Patent # 6,634,045
Other Patents Pending

Gently raise both legs of the patient and place the HeelZup® Cushion under the legs with both heels fully suspended. Place a reusable incontinence pad or an empty pillow case on the calf bed of the disposable, bare foam version of the HeelZup® Cushion to protect the foam from soils and moisture.

Please be sure to avoid placing the cushion directly behind the knees.



For more leg elevation or to enhance patient comfort, place a conventional pillow on top of the HeelZup® Cushion. The raised borders will hold the pillow in place, preventing lateral movement. Always inspect for skin integrity every two hours.

To clean Reusable HeelZup® Cushions, wipe down the moisture-proof nylon cover with any properly diluted disinfectant solution according to manufacturer's instructions. You may also use a 10:1 bleach water solution. Allow to air dry before using.

Please do not machine launder the cover or place it in a dryer.



Intensive Therapeutics, Inc.
P.O. Box 488
Sarasota, FL 34230

Questions? Contact your authorized HeelZup® distributor or call Intensive Therapeutics, Inc.
toll free at **866-736-8484**.